Little Doctor

LD-812U

ENGLISH

ULTRASONIC NEBULIZER LD. *Instruction Manual.*

ROMANIAN

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ENG	Electric power socket
PARTS AND COMPONENTS (fig.1)	Batteries
● Nebulization chamber	Mouthpiece
● Medication cup	Mask for adults
● Mesh membrane	Mask for children
● Electrodes	Carrying bag
● Main unit	Power cord
● Button	
● Indicator	
● Battery compartment	

MESH TECHNOLOGY OF AEROSOL GENERATION
LD-812U is a nebulizer that uses a new mesh technology of aerosol generation. Compared to conventional ultrasonic nebulizers, the use of the mesh technology gives a number of advantages:
• super compact size of the device
• use of a wide range of drugs, including antibiotics, antiseptics and mineral water as inhalation solutions;
• remaining volume of the inhalation solution is reduced to almost zero, which helps save expensive drugs;
• the device can work from either a USB-adaptor (not included) or 2 AA batteries;
• device operation is virtually silent and inhalation therapy can be administered even to sleeping children;
• you can tilt the device during inhalation up to 45° from the vertical axis.
• self-cleaning membrane significantly extends the service life of the nebulization chamber
• small particle size provides for more efficient delivery of medication to the lower respiratory tract
• two masks (adult mask and children mask) and two mouthpieces are in the package.

OVERVIEW
The purpose of LD Ultrasonic Nebulizer is to treat and prevent respiratory and lung disorders using aerosols of AQUEOUS solutions of drugs in hospitals and at home. This manual will help the user to operate LD Ultrasonic Nebulizer safely and efficiently. Use the device shall in accordance with the rules in this manual and do not use it for purposes other than those described herein. Make sure you read and understand the entire manual. Functionally, the device consists of an aerosol formation chamber with a mesh membrane and a main unit with a battery compartment. There is a button and a light indication around the button on the case of the device.

SAFETY WARNINGS
Caution! Never use inhalation solutions containing ether, oils or suspended particles (suspensions), including decoctions and infusions of herbs. We recommend to use any types of standard liquid inhalation solutions designed for nebulizer therapy produced by pharmaceutical companies, and natural mineral waters.
Caution! To avoid damage, do not touch the mesh membrane with your hands, do not clean it with any objects, alcohol or solvents.
• Inhalation solutions should be prepared under sterile conditions based on 0.9% sodium chloride as a solvent. Do not use tap or even boiled water as an inhalation agent. Disinfect the vessel where the solution is to be prepared by boiling.
• Type of inhalation (through the mouth, using a mouthpiece, or through the nose using a mask), the duration (usually no more than 10–15 minutes) and frequency, as well as the inhalation solutions used should be determined by YOUR DOCTOR.
• Children shall use the device under adult supervision.
• If you do not use the device for a long time, remove the batteries.
• Do not place the device in water, under flowing water drain or in the shower. Do not use it when bathing.
• If the device does not work, see the "TROUBLESHOOTING ALGORITHM AND METHODS" section.
• Only use accessories that are intended for LD-812U and are described in this manual.
• Do not insert foreign objects into the holes of the main unit.

BATTERY INSTALLATION
The package of nebulizer has two AA batteries. The included batteries are designed to test the performance of the device, and their service life may be shorter than that of new batteries. To install batteries, open the battery compartment by pressing the cover latch with your finger (fig. 2). Insert the batteries, observing the polarity (fig. 3). Close the battery compartment cover by pressing your finger until it clicks. New batteries, depending on the type and capacity, can ensure the operation of the nebulizer for an average of 6 days (when the nebulizer is working for 15 minutes a day). If the indicator is constantly lit in orange it means that the battery is low and inhalator will stop working. Replace the batteries with new ones.
• Do not leave used batteries inside the device.
• We recommend to use alkaline AA batteries to power the device.
• Zinc-carbon batteries are not recommended.

THE USE OF THE DEVICE WITH A POWER SUPPLY
Use a power supply (not included) with the following specifications.
Output voltage: 5V ± 5%
Load current: at least 500 mA
Plug: micro-USB
Find the power supply socket on the back case of the device. Connect the power cord to the main unit, plug the power cord into electrical wall outlet and press the button. After measurement, switch the device off by pressing the button, disconnect the power source from the electrical wall outlet and disconnect the power cord from the device.

OPERATING PROCEDURES
Important! Before using the device for the first time, clean it as described in paragraph 1 of section "Care, storage, repair and disposal".
1. Insert the inhalation chamber into the slots in the upper part of the main unit until it clicks (fig. 4).
Attention! Keep the electrodes of the device and nebulizer chamber clean. Prevent water or drugs from getting on them! Otherwise, malfunction of the inhale is possible.
Important! The nebulization chamber with a mesh membrane are consumable materials. The service life of the nebulization chamber varies depending on the usage environment and the composition of the inhalation solutions used. Under proper operation conditions and provided that the device is used to nebulize a saline solution 3 times a day for 10 minutes at a room temperature (23 °C) the service life of the nebulization chamber is one year (amount 180 hours).
2. Open the top of the nebulization chamber by lifting the latch (fig. 5). Pour the medication into the medication cup. The maximum capacity of the medication for one inhalation is 8 ml (see the scale on the nebulization chamber), the minimum is 0.5 ml. Close the cover of the nebulization chamber by lowering the latch. Do not use excessive force for opening and closing the latch (thus you will increase the service life of the nebulizer chamber). Ensure that the cover is tightly closed and the inhalation solution does not spill out of the nebulizer chamber.
3. Insert the mouthpiece (fig. 6) or mask (fig. 7).

Attention, parents! While using of the device during inhalation the child can breathe simultaneously through the mouth and through the nose. The design of the mask prevents the formation of excessive pressure of the aerosol during the inhale, which allows the child to breathe more calmly, evenly and deeply.
4. For the beginning of the procedure press the button. The indicator lights up with green color.
Attention! If the nebulizer chamber is empty when the power is on, the indicator light will glow green for one second, and then the device will switch off automatically. The nebulizer may spray the inhalation solution for one second after the power is on, and then pause for 0.5 seconds. This is normal operation of the device. After a pause, the device should spray the inhalation solution continuously.

INHALATION
For effective inhalation, calm down, relax, and sit up. If you inhale in bed, place a pillow under your back so that your back is as straight as possible. An inconvenient posture and uneven breathing may lead to blocking a part of the lungs and some parts of the respiratory tract.
Important! A single-patient use of mouthpiece and/or mask is recommended. Take a comfortable position and begin the inhalation procedure. Breathing should be even, breathe slowly and deeply so that the aerosol penetrates deeply into the respiratory tract.
Attention! During inhalation, do not close the openings (fig. 8). This will reduce the efficiency of the device. If you want to stop the procedure, press the button to turn off the power. The indicator goes out. If the inhalation solution is fully used, the device will automatically switch off.
Attention! If during the inhalation the device does not switch off automatically after the entire inhalation solution has been used, turn off the power by pressing the button. During the procedure, you can tilt the inhale (at an angle of no more than 45° relative to the vertical axis). However, make sure that the inhalation solution is in contact with the mesh membrane. The inhale may operate normally for some time after changing the angle of inclination of the device. If the angle of inclination of the inhale is such that the inhalation solution is not in contact with the mesh membrane, it will work normally for about 10 seconds and then switch off (the operation time depends on the type of inhalation solution used). When a small amount of the inhalation solution is left, tilt the device towards you. Thus you will use the remaining inhalation solution completely. Do not shake the inhale during use. It may automatically switch it off. After inhalation, switch off the device by pressing the button, the indicator will go out. Remove the inhalation chamber from the device by pressing the PRESS button on the back of the case and pushing the nebulization chamber forward (fig. 9). Clean the device as described in paragraph 1 of section "Care, storage, repair and disposal".

MEMBRANE CLEANING MODE
The membrane cleaning mode is designed for automatic cleaning of the membrane from residues of drugs and other deposits. To enable the membrane cleaning mode:
1. Pour 8 ml of distilled water into the medication cup.
2. In the off state, press and hold the button for about 5 seconds until the red indicator lights up. Release the button.
3. After about 3 to 5 seconds, the indicator goes out and the membrane cleaning procedure begins.
4. The device will switch off automatically after 10 minutes. For the forced shutdown of the membrane cleaning mode briefly press the button. Forced shutdown is possible after 3 seconds after switching on. We recommend to clean the membrane at least 1 time per month with a daily use of the device or in cases where the aerosol generation has decreased or it is completely absent.

CARE, STORAGE, REPAIR AND DISPOSAL
1. After an inhalation procedure with any medicinal solution, it is recommended to spray pure water for 1-2 minutes in order to clean the mesh membrane from drug residues. DO NOT clean the membrane with any objects (napkins, rags, cotton swabs, etc.).

2. Clean the device and accessories regularly. It is recommended to use all accessories of the device with a 3% solution of hydrogen peroxide with the addition of a 0.5% solution of detergent (for example, laundry detergent). Then wash the nebulization chamber and medication cup under running water. Mouthpieces can be boiled for 10 minutes or autoclaved at a temperatures up to 150°C. After processing, wipe all parts of the device with a soft cloth. Attention! After cleaning, do not install the nebulization chamber with residual water on the main unit — this may lead to oxidation of the contacts. Wait until the camera is completely dry.
3. Protect the device from direct sunlight and strong shocks.
4. Do not store or use the device in the immediate vicinity of heaters and open flame.
5. Protect the instrument from contamination.
6. Do not allow the device to come in contact with aggressive solutions.
7. If necessary, carry out repairs only in specialized organizations.
8. This device has a service life of 5 years from the moment of its transfer to the consumer. For consumables - 1 year from the date of transfer to the consumer. At the end of the specified service life, periodically contact specialists (specialized repair organizations) to check the technical condition of the device and, if necessary, to carry out its disposal in accordance with the rules of disposal applicable in your region. No special conditions for disposal by the manufacturer are established.

Green light	power on
Orange light	low battery
Flashing green light	membrane cleaning mode is on

BASIC SPECIFICATIONS	
Model	LD-812U
Type	ultrasonic
Power consumption, not more, W	3
Nebulization rate, not less, ml/min (Aerosol Output Rate)	0.2*
Median mass aerodynamic diameter (MMAD), not more, µm	5*
Capacity nebulization chamber, ml	8
Remaining capacity of the inhalation solution, not more, ml	0.5
Ultrasonic frequency, kHz	110
Noise level, no more than, dB	40*
Power source	3V (2xAA) or DC 5V/0.5A
Electric power socket	micro-USB
Degree of patient contact	type BF
Electrical safety	II
Low battery indication	1.9V ±0.2V
Operation from one set of alkaline batteries, min	about 90
Operation conditions: temperature, °C	10 to 40
relative humidity, % Rh	15 to 85
Storage and handling conditions: temperature, °C	minus 20 to 40
relative humidity, % Rh	15 to 85
Weight (the main unit and nebulization chamber), no more than, g	90
Dimensions of the main unit with nebulization chamber, mm	(73 ± 2) × (99 ± 2) × (45 ± 2)

Year and month of production
Indicated on the device case in the serial number in the form of «YYMM812XXXX», where YY is the year and MM is the month of production.
SYMBOL EXPLANATION:
CE Conforms with the Directive 93/42/EEC
! Important: Read the instruction
! BF type product
IP22 IP protection class

Production of the devices is certified according to the international standard ISO 13485.
The device complies with European Directive MDD 93/42/EEC.
The revision date of this Operation manual is indicated at the last page in the EXXX/YYMM/NN format, where YY is the year, MM is the month, and NN is the revision number. Technical specifications are subject to change without prior notice in order to improve operational properties and quality of the product.

Problem	Cause	Recommended actions
The intensity of aerosol formation is very low.	There is no contact between the inhalation solution and the mesh membrane for more than 10 seconds. Mesh membrane inhalation chamber is clogged. Use of non-recommended high viscosity drugs.	Change the tilt angle of the nebulizer so that the inhalation solution is in contact with the mesh membrane. Clean the membrane as described in the MEMBRANE CLEANING MODE section. If after cleaning the problem persists, replace the inhalation chamber with a new one. Wipe away any contamination and turn the power on again.
Electrodes of the nebulizer and inhalation chamber are contaminated.		Wipe away any contamination and turn the power on again.
The indicator is orange, the battery level is insufficient.		Replace all batteries with new ones and turn on the power again. Try a different power adapter / power cord

After power on, the indicator flashes in green and then turns off.	Nebulizer chamber is empty.	Pour the inhalation solution into the inhalation chamber.
There is no contact between the inhalation solution and the mesh membrane.		Adjust the tilt angle of the nebulizer so that the inhalation solution is in contact with the mesh membrane.
Electrodes of the nebulizer and inhalation chamber are contaminated.		Wipe away any contamination and turn the power back on.

The indicator does not light, and the spraying of the drug does not occur.	Incorrectly installed batteries.	Install the batteries, observing polarity, and turn on the instrument again.
Low battery charge.		Replace all batteries with new ones and turn on the power again.
The indicator lights up but no spraying occurs.	The indicator lights up orange - very low battery charge. Disruption of the mesh membrane nebulizer chamber.	Replace all batteries with new ones and turn on the power again. Replace the inhalation chamber with a new one.
Electrodes of the nebulizer and inhalation chamber are contaminated.		Wipe away any contamination and re-power on.
The mesh membrane of the inhalation chamber is heavily clogged.		Clean the inhalation chamber as described in the "Care, storage, repair and disposal" section. If after cleaning the problem persists, replace the inhalation chamber with a new one.
Nebulizer turns off during operating.	Out of inhalation solution.	Pour an inhalation solution to the nebulizer chamber.
There is no contact between the inhalation solution and the mesh membrane for more than 10 seconds. Nebulizer was shaken during use.		Change the tilt angle of the nebulizer so that the inhalation solution is in contact with the mesh membrane. Hold the nebulizer in your hand steadily, do not shake it.

Inhalation solution leaks from the nebulizer chamber.	Disruption of the mesh membrane of the inhalation chamber or aging of the seal of the inhalation chamber.	Replace the inhalation chamber with a new one.
Mesh membrane inhalation chamber is clogged.	Mesh membrane is clogged with drug residues and other deposits.	Clean the membrane as described in the MEMBRANE CLEANING MODE section.

WARRANTY
The warranty liabilities are contained in the warranty card given at the sale of this device to a purchaser. The warranty does not apply to the consumables (inhalation chambers, masks, mouthpieces, etc.). The addresses of organizations providing warranty service can be found in section "INFORMATION ON THE MANUFACTURER AND DISTRIBUTORS" or on the website www.littledoctor.sg.
INFORMATION ON THE MANUFACTURER AND DISTRIBUTORS
Manufactured under control and for Little Doctor International (S) Pte. Ltd. (7500A BEACH ROAD #11-313 THE PLAZA SINGAPORE 199591. Postal address: Yishun Central P.O. Box 9293 Singapore 917699).
Manufacturer: Little Doctor Electronic (Nantong) Co. Ltd. (No. 8, Tongxing Road Economic & Technical Development Area, 226010 Nantong, Jiangsu, People's Republic of China).
Distributor in Europe: Little Doctor Europe Sp. z o.o., 57 Zawiłia Street, 30-390, Krakow, Poland, Sales Office phone: +48 12 2684746, fax: +48 12 2684 47 53. E-mail: biuro@littledoctor.pl, www.LittleDoctor.pl

Authorized Representative in the EU: Little Doctor Europe Sp. z o.o., 57 Zawiłia Street Krakow 30-390 Poland.
For more information please visit www.littledoctor.sg

* - Measured by Little Doctor International (S) Pte. Ltd.

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