Little Doctor

LD Cardio

ENG

STETHOSCOPE LD. *Instruction Manual.*

POL

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HUN

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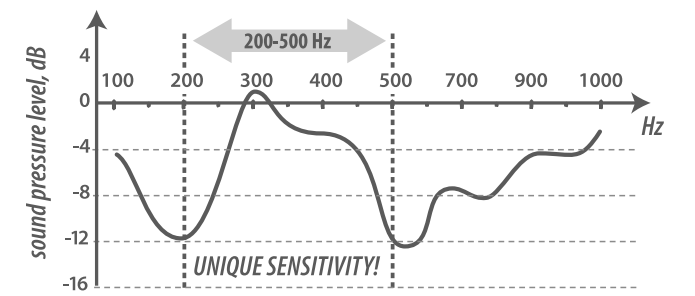
Membrane for stethoscope.
- 3

Plastic ring.
- 4

Connecting medical tube with internal spring and built-in binaural tube.
- 5

Ear tips (stud olives), soft.
- 6

Nameplate.
- 7

Additional set of accessories.
- 8

Instruction Manual + Warranty Card.

## GENERAL INFORMATION

LD Cardio – high-quality cardiologic stethoscope for physicians. LD Cardio enables to carry out auscultation within the whole frequency range of adults and children. Special geometry of head and membrane material make it possible to emphasize attention during auscultation to medium range, which is especially important during auscultation of heart sounds. Frequency characteristic of stethoscope within 200-500 Hz has sensitivity – unique for stethoscopes of this class (fig.2). LD Cardio has high-quality acoustic system of acoustic transmission from membrane to ears of physician. The stethoscope head and sound-conducting tube are provided with two sound-conducting ducts, separately for each ear. Both sound-conducting ducts are integrated into one whole to exclude friction sounds from each other. Material of sound-conducting tube, optimal rigidity of attached ear tips and accuracy of connection of stethoscope parts – exclude at maximum foreign and self-noise during its use.

| Nº | Name of part                           | Quantity | Model   | Comment                                         |
|----|----------------------------------------|----------|---------|-------------------------------------------------|
| 1  | Stethoscope head                       | 1 pc.    | LD-S044 | with membrane                                   |
| 2  | Connecting medical tube                | 1 pc.    | LD-S043 | with internal spring and built-in binaural tube |
| 3  | Ear tips (stud olives), soft           | 1 pair.  | LD-S046 |                                                 |
| 4  | Ear tips (stud olives), soft           | 2 pairs. | LD-S046 | enters additional set of accessories            |
| 5  | Membrane                               | 1 pc.    |         | enters additional set of accessories            |
| 6  | Case for additional set of accessories | 1 pc.    |         | enters additional set of accessories            |
| 7  | Nameplate                              | 1 pc.    |         |                                                 |
| 8  | Instruction manual + Warranty Card     | 1 pc.    |         |                                                 |
| 9  | Package                                | 1 pc.    |         |                                                 |

## TECHNICAL SPECIFICATIONS

|                                                                                   |                                     |
|-----------------------------------------------------------------------------------|-------------------------------------|
| Model                                                                             | LD Cardio                           |
| Diameter of membrane                                                              | 44 mm                               |
| Length of tube                                                                    | 56 cm                               |
| Operation conditions: temperature, °C<br>relative humidity, % Rh                  | from 10°C to 40°C<br>85 and lower   |
| Storage and transportation conditions: temperature, °C<br>relative humidity, % Rh | from -34°C to +50°C<br>85 and lower |
| Size with package, mm                                                             | 440 x 180 x 43                      |
| Weight (without package)                                                          | 200 g                               |
| Year of manufacture                                                               | printed on package                  |

### SYMBOLS:

- CE

Compliance with Directive 93/42/EEC
- Important: Read the manual

Important: Read the manual
- Manufacturer

Manufacturer
- Protect from moisture

Protect from moisture
- Representative in the European Union

Representative in the European Union

The revision date of this Instruction Manual is shown on the last page as EXXX/YYMM/NN, where YY is the year, MM is the month, and NN is the revision number.

## CARE, STORAGE, REPAIR AND DISPOSAL

- Keep this Stethoscope from exposure to higher humidity, direct sunlight, shocks.
- Do not keep and use this Stethoscope near heating installations and open fire.
- Keep the Stethoscope clean and protect it from dust.
- Do not allow contact with corrosive solutions.
- Repair the Stethoscope only in authorized organizations, addresses on Warranty Card.
- Dispose of the Stethoscope and its components according to the application local regulations. No special requirements to disposal of this Unit are defined by the manufacturer.

## WARRANTY

- The following LD product is covered by warranty for the period specified in the warranty card.
- The addresses of organizations providing warranty service can be found in section “MANUFACTURER AND AUTHORIZED REPRESENTATIVES” or on the website [www.littledoctor.sg](http://www.littledoctor.sg).

## CERTIFICATION AND STATE REGISTRATION

This unit manufacturing is certified according to international standard ISO 13485. Stethoscope comply with the requirements of European Directive MDD 93/42/EEC.

**Manufactured under control and for** Little Doctor International (S) Pte. Ltd. (7500A BEACH ROAD #11-313 THE PLAZA SINGAPORE 199591. Postal address: Yishun Central P.O. Box 9293 Singapore 917699).

**Manufacturer:** Little Doctor Electronic (Nantong) Co. Ltd. (No. 8, Tongxing Road Economic & Technical Development Area, 226010 Nantong, Jiangsu, People’s Republic of China).

**Distributor in Europe:** Little Doctor Europe Sp. z o.o., 57G Zawila Street, 30-390, Krakow, Poland, Sales Office phone: +48 12 2684746, 12 2684747, fax: +48 12 268 47 53. E-mail: [biuro@littledoctor.pl](mailto:biuro@littledoctor.pl) [www.LittleDoctor.pl](http://www.LittleDoctor.pl)

**Authorized Representative in the EU:** Little Doctor Europe Sp. z o.o., 57G Zawila Street Krakow 30-390 Poland.

For more information please visit [www.littledoctor.sg](http://www.littledoctor.sg)

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