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 Little Doctor Electronic (Nantong) Co. Ltd.,
No.8, Tongxing Road Economic & Technical Development
Area, 226010 Nantong, Jiangsu, People's Republic of China

EC REP LITTLE DOCTOR EUROPE SP. Z O. O.,
57G Zawila Street Krakow 30-390 Poland.

© Registered Trade marks of Little Doctor International (S) Pte. Ltd.
© Little Doctor International (S) Pte. Ltd., 2000-2020

 **DOWNLOAD
USER MANUAL**
www.littledoctor.sg/manuals

B084/2009/15

BGR Маншет LD-CUFF.
CZE Manžeta LD-CUFF
ENG Blood pressure LD-CUFF
HUN LD-CUFF mandzsetta
KAZ LD-CUFF көмкермесі

POL Mankiet LD-CUFF
ROU Manșeta LD-CUFF
RUS Манжета LD-CUFF
UKR Манжета LD-CUFF

WITHOUT RING						
	7-12 cm		N NEONATAL	COTTON	C1N	C2N
	11-19 cm		I INFANT	COTTON	C1I	C2I
	18-26 cm		C CHILD	COTTON	C1C	C2C
				NYLON	N1C	N2C
	25-40 cm		A ADULT	COTTON	C1A	C2A
				NYLON	N1A	N2A
D-RING	34-51 cm		L LARGE ADULT	COTTON	C1L	C2L
				NYLON	N1L	N2L
	40-66 cm		T THIGHT	COTTON	C1T	C2T
	25-36 cm		A ADULT	NYLON	N1AR	N2AR
D-RING	33-46 cm		L LARGE ADULT	NYLON	N1LR	N2LR

Little Doctor International (S) Pte. Ltd.
Yushun Central P.O. Box 9293, Singapore 917699 Fax: (65) 62342197 E-mail: info@littledoctor.sg

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ENG Cuff model LD-CUFF for LD-series blood pressure monitors. Range of measurement of pressure 0-300 mmHg. Conditions of operation: temperature +10°C +40°C; humidity 85% or below. Conditions of transportation/storage: temperature - 34 °C + 65 °C; humidity 85% or below. Protect the bladder and tubes from sharp objects. The cuff is resistant to multiple sanitization. Use only a soft, dry cloth to clean the unit. Do not use gasoline, paint thinner, or other strong solvents on the unit. Since the cuff may absorb perspiration and other fluids, inspect it

for stain and discoloration after each use. When cleaning the cuff, do not machine wash or scrub it. Use a synthetic detergent and gently rub the surface. Warranty time 6 months from the date of purchasing. Operating life 3 years since transfer to the consumer.. After the expiration of the term lifetime is necessary periodically apply to specialized service centres to verify the possibility of further use. During recycling follow your local rules. The producer does not prescribe the special conditions of recycling for this device. Year of manufacture indicated on the label. Product of Little Doctor International (S) Pte. Ltd., Yishun Central P.O. Box 9293 Singapore 917699. Distributor in Europe: Little Doctor Europe Sp. z o.o. 57G Zawila Street, 30-390, Kraków, Poland. Sales Office phone: +48 12 2684746, 12 2684747, fax: +48 12 268 47 53. E-mail: biuro@littledoctor.pl www.LittleDoctor.pl

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